
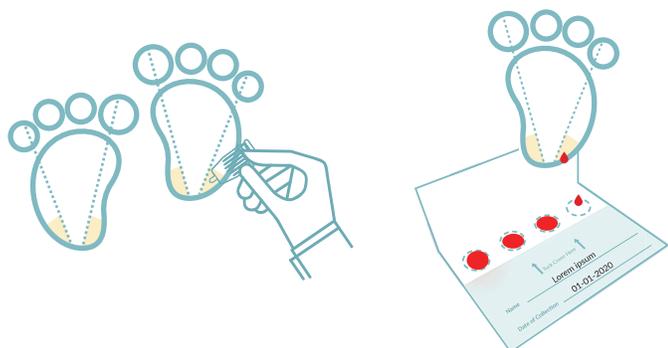


*For your convenience we have created a separate set of collection instructions for young children. For complete instructions, please refer to each specific Vibrant Specimen Collection instruction document.

Gut Zoomer Test

Stool collections can be collected from child's diaper or from a child's training toilet. Please avoid added debris from the diaper in the collection sample. Make sure stool sample does not come in contact with the toilet water or urine. Slowly drain urine away from bowel movement before collecting a sample if necessary. Please ensure no chemicals are used to clean the child's toilet prior to collection.

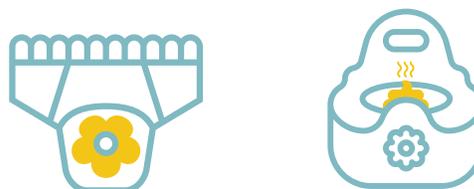

Dried Blood Spot

To collect blood for a blood spot card please follow the instructions on the Dried Blood Spot Collection Instruction Booklet. Blood can also be collected from the heel instead of the finger by sticking the outer or inner curve of the heel, as shown in the cross-hatched areas on the heel below (do not collect blood from the center of the heel). Allow the drops of blood to fall onto the card to fill the circles.

Urine Collection

Collect the first morning urine in urine collection cup. Collecting the sample prior to eating or drinking is recommended. A urine collection container called a "hat" can be used inside any toilet seat to collect urine. If the child is not potty trained, Medline Sterile Pediatric Urine Collector bags can be used to collect urine. If unable to use these options, Curad Sterile Cotton Balls can be placed in a child's diaper and urine can be extracted by squeezing the urine out of the cotton balls into the urine collection cup.

